

CARDIOKEY™

CARDIAC MONITORING QUICK START GUIDE

STEP 1: SKIN PREPARATION



Very Important:
Remove the ECG Prep Kit and **follow the skin preparation steps on pages 4-5** of the Patient Education Guide.

STEP 2: REMOVE SUPPLIES




Remove CardioKey, lanyard (optional), lead wires, and electrodes from the kit.

STEP 3: ATTACH ELECTRODES



Select two electrodes and snap lead wires onto electrode patches.

STEP 4: ATTACH TO CHEST

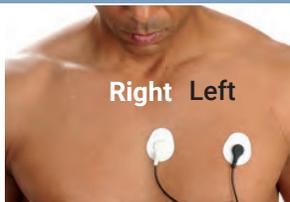


As shown place electrode patches with lead wires attached to your body.

STEP 5: CONNECT CARDIOKEY



Fully insert device into lead wires and press  button for a "BEEP" to ensure data is recording.

STEP 6: MONITORING

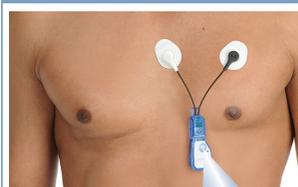

Cardiokey is now capturing every heartbeat. To record a symptom, press the heart shaped button.

PLEASE READ THE PATIENT EDUCATION GUIDE FOR COMPLETE INSTRUCTIONS.

FOR ASSISTANCE, CONTACT SUPPORT: 1-877-593-6421

AT THE END OF SERVICE, PLEASE DO THE FOLLOWING:

- Please verify that your name and ID sticker from the kit have been added to the front of the patient diary.
- Place the CardioKey, lead wires, and completed diary in the self-addressed, postage-paid envelope supplied in the kit.
- The prepaid envelope can be placed into your home mailbox or dropped off at any USPS Post Office location.



CARDIOKEY™

CARDIAC MONITORING QUICK START GUIDE

Quick Tips for Your Monitoring Period



Wear CardioKey only for as long as prescribed by your doctor.

Your prescription ends on _____



CardioKey is **NOT waterproof and must be removed before showering, bathing or swimming**. Disconnect CardioKey and the lead wires from the electrodes. You can shower with the electrodes on.



Remember to **press the raised button** on CardioKey to record an event and **write any and all symptoms** into your patient diary.



Place CardioKey, lead wires and the Patient Diary into the supplied self-addressed, postage paid envelope and take to your local post office. **Ensure your name and a kit identification sticker has been affixed to the patient diary.**

Contact us at **1.877.593.6421** if you need assistance or if you have questions throughout your monitoring period.

Customer Service is available:

Monday – Fridays: 8:00 AM - 8:30 PM ET

Saturdays: 8:00 AM - 4:30 PM ET.

1000 Cedar Hollow Road, Malvern, PA 19355
1-866-426-4402 | gobio.com

